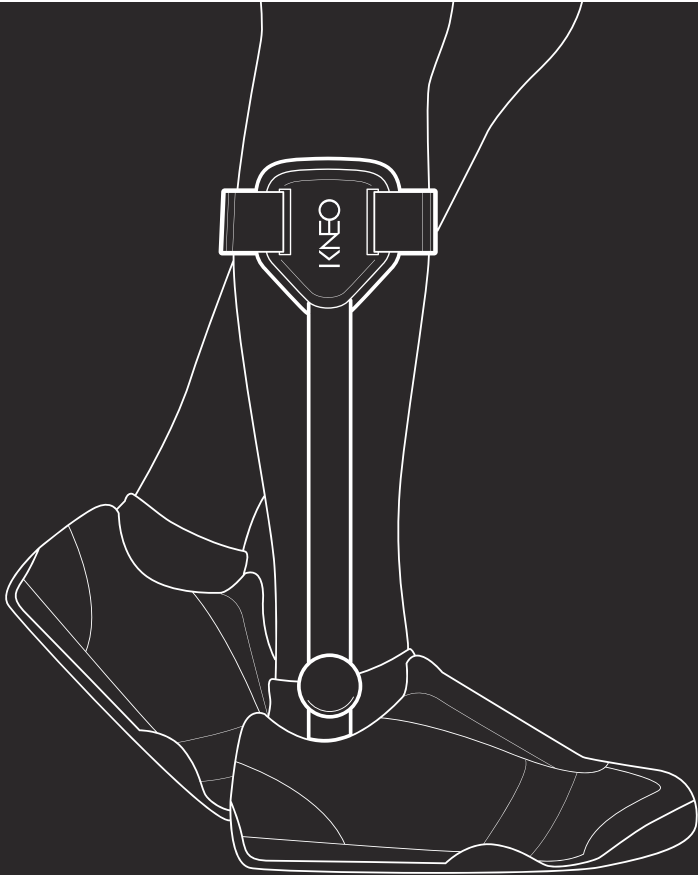
KNEO

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Datum/Revision Januar 2013

ENGLISH 1. Purpose

The knee brace is intended for medical use only at the knee joint and solely for contact with intact skin.

Note: The brace is intended for use by one patient only.
Do not reuse.

2. Indication and operation

Use in:

- Osteoarthritis of the knee, especially with bow legs or knock-knee deformity
- Injuries to the knee joint, in which the meniscus needs to be relieved
- After surgery on ligaments and meniscus to protect the surgical outcome
- After collateral ligament injuries of the knee, especially when instability is present

The indication is provided by the physician.

The knee brace works by shifting the load line passing through the knee joint medially or laterally as needed. The other compartment is correspondingly relieved. The forces needed to relieve the stress on the meniscus are generated by standing with an outstretched leg.

3. Selection of the orthosis

The brace is available in four sizes. These differ in the ankle height and the lower leg bar length.

Identification / REF	Ankle height in cm	Lower leg bar length in cm
2031001-07 L/R	7 cm	27 cm
2031001-08 L/R	8 cm	30,5 cm
2031001-09 L/R	9 cm	34 cm
2031001-10 L/R	10 cm	37,5 cm

L = left, R = right.

4. Design description

The knee brace consists of a foot piece and a side rail forming a joint in the ankle area. Attached to the end of the side rail is a movable pad that is equipped with a fleece cushion and is fastened to the lower leg with a hook-and-loop fastener.

5. Adaptation of the brace to the patient

Adapting the brace to the patient's shoe, the foot plate of the orthosis is cut to fit (Fig. 2–5). The ankle position (Fig. 3) must be taken into account. The size markings on the foot plate are to serve as a guide only. Furthermore, the foot plate has to be bent into the contour of the insole. The position of the ankle joint of the brace must line up with the patient's ankle joint in the shoe. This is achieved by cutting the foot plate of the brace accordingly. The foot plate can be positioned under the insole (Fig. 5). The shoe is an integral part of the knee brace, which provides the position and function.

Note: Using a solid soled, lace-up shoe is recommended.

The lower leg bar must match the lateral contour of the lower leg (Fig. 6).

- For medial osteoarthritis it is necessary to apply lateral pressure below the knee. This is achieved through the proper adjustment of the ankle and lower leg bar (Fig. 7). Ensure that the pressure generated on the leg meets the patient's requirements.
- For lateral osteoarthritis it is necessary to apply medial pressure below the knee. This is achieved by the knee brace pulling from the lateral side.

Note: Checking the effect of orthotic correction is recommended. This can be done by measuring the tension, or with the LASAR Posture from Otto Bock.

The hook-and-loop fastener is shortened as required at the fixed end (Fig. 8). After proper fitting, the hook fastener is replaced by the mushroom fastener. Alternatively, there is the possibility of incorporating an elastic strap (Figs. 15–18).

To use the elastic hook-and-loop fastener fasten it through the rear eyelet of the pad as shown (Figs. 15 + 16) and put it together with the main hook-and-loop fastener to fit the lower leg (Figs. 17 + 18).

To apply the heat shrinking tube the fleece cushion must be removed (Fig. 9). Remove the rail stop from the pad (Fig. 10) and take it from the bar. The caps on the joint are removed (Fig. 11) and the heat shrinking tube is heated until a snug fit over the joint is obtained (Fig. 12). The outer end is cut to the required length of about 57 mm (Fig. 13).

The foot pad is to be positioned on the lower leg bar with the prepared double-sided tape as needed. Fix it with the cable tie and cut it into right shape (Fig. 14).

6. Operating and Maintenance

- Examine the brace device before each use.
- If any part of the brace has significant wear, for example, excessive play in the joint, the brace is no longer usable. Request replacement parts from CHW Technik GmbH.
- The fleece cushion and the straps are removable and can be cleaned with mild soap and air dried.
- If required, it can be treated with a commercial spray disinfectant.

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The CE mark indicates compliance with the relevant requirements of the Medical Device Directive 93/42/EEC

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Lagerset · Bearing washers

- 2031204 Lagerset

Gelenkabdeckkappen · Joint cover caps

- 2031205 Gelenkabdeckkappen

Gelenkschraube · Joint bolt

- 2031206 Gelenkschraube

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Fußpolster · Foot pad

- 2031208 Fußpolster

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CHW-Technik GmbH