



**CHW-Technik**  
OT-Produkte GmbH

# Instructions Clamping Aid 6020003



The clamping aid is intended for fixing models at the respective workstation for further processing.

It is clamped horizontally in the vise. The pin is on the vise and the screw is up.



The plaster is inserted with the cast-in rod into the clamping aid and screwed with the thumbscrew into the desired position.

By pivoting the vise the workpiece can also rotate horizontally.



With the clamping of the vise the plaster can be turned around the pin. When loosening the vise, the gripping aid rests on the pin and the plaster must be held only on the other side.





The clamping aid has a 25 mm bore and is suitable for rods from a diameter of 10 mm.



CHW-Technik GmbH  
Kolligsbrunnen 1, 37115 Duderstadt  
Tel.: 05527/998969  
Fax.: 05527/998967  
[h.wagner@chw-technik.de](mailto:h.wagner@chw-technik.de)  
[www.chw-technik.de](http://www.chw-technik.de)

Date / Revision 2022-12